

## Intended use

For the collection of nucleic acids from saliva samples.

## Precautions

**Do not eat, drink, smoke, brush your teeth or chew gum for 30 minutes before use.** Check fluid level is at 2ml before opening. If solutions appear cloudy before use, warm tube between hands until solution becomes clear.

- Remove the cap from the tube. Keep cap for later use. Screw collection funnel on to the tube.
- Spit into the funnel until the level of the liquid in the tube, not including bubbles, reaches the 4ml line. Delivery of saliva will take between 2 and 5 minutes.
- Unscrew the collection funnel and replace the tube cap tightly. Discard the funnel.
- Shake the collection tube several times to mix the saliva with the solution.

## Contents

Funnel, Tube Pre-Filled with Stabilization Buffer, Cap.



### Precautions

Do not swallow. If stabilizing solution comes into contact with skin or eyes, wash with plenty of water. Tube cap may be a choking hazard to small children. MSDS available at [www.isohelex.com](http://www.isohelex.com)